

maxill
1-800-268-8633

DIN 02282461

Unscented
ITEM 61163RU

maxill



HARD SURFACE DISINFECTANT
READY TO USE TOWELETTES • CLEANS AND DEODORIZES

**THE 1 MINUTE TUBERCULOCIDE, BACTERICIDE
VIRUCIDE & FUNGICIDE FOR WASHABLE HARD SURFACES**

- Highly Flammable
- Store away from heat sources
- Do not expose to an open flame
- Keep out of reach of children
- For external use only

Active Ingredients


Ethyl Alcohol79.0% w/w
Benzalkonium Chloride..... 0.2% w/w
o-Phenylphenol 0.1% w/w



INSTITUTIONAL USE ONLY • READ THE LABEL BEFORE USING



**REFILLS
160**
REFILL TOWELETTES

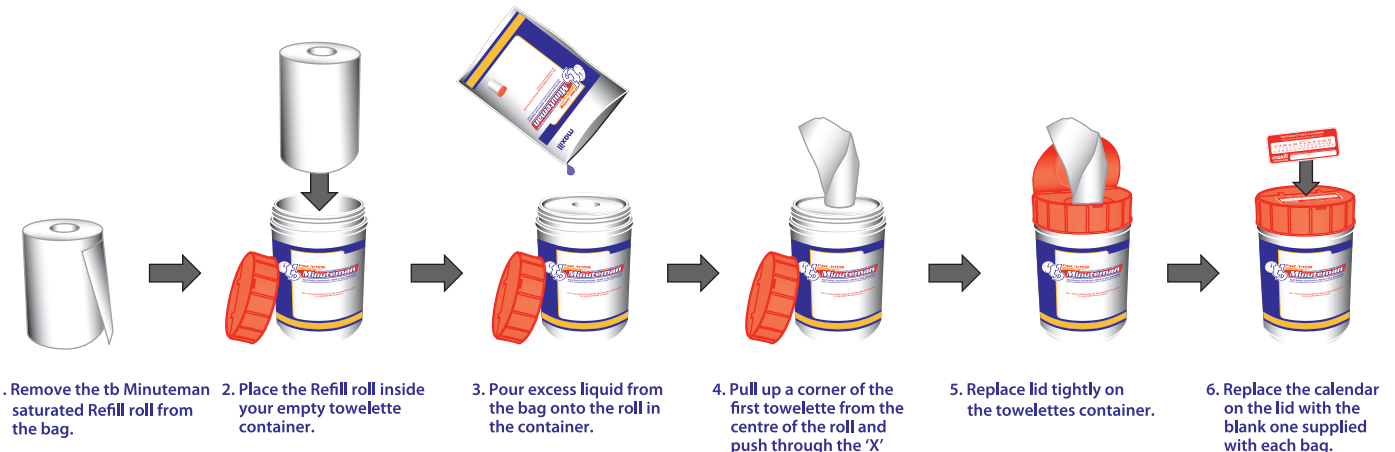

Product Description

tb Minuteman is an intermediate level hard surface disinfectant as well as a tuberculocide, virucide and fungicide. **tb Minuteman** is designed for use in dental and medical clinics, hospitals and nursing homes where disinfection and prevention of cross-contamination is of prime importance. **tb Minuteman** is safe for use on non-porous environmental hard surfaces, such as counters, trays, tables, etc. **tb Minuteman** will not stain or leave residual smears after drying. **tb Minuteman** is not for use in hospital nurseries.

Product Information

tb Minuteman provides complete tuberculocidal activity against:

- Mycobacterium bovis (BCG)

tb Minuteman provides complete bactericidal activity against:

- Pseudomonas aeruginosa
- Salmonella choleraesius
- Staphylococcus aureus
- Methicillin Resistant Staphylococcus aureus (MRSA)
- Vancomycin Resistant Enterococcus faecalis (VRE)

tb Minuteman provides complete fungicidal activity against:

- Trichophyton mentagrophytes

tb Minuteman provides complete virucidal activity against

- Influenza A virus
- Poliovirus type 1, Sabin
- Herpes simplex virus type 1
- Herpes simplex virus type 2
- Human Immunodeficiency virus type 1 (HIV-1)*

Directions for Use

Preclean the surface by wiping with a towelette to remove organic soil. Dispose of towelette. To disinfect the precleaned surface, apply **tb Minuteman** liberally by wiping the surface with a towelette until the surface is thoroughly wet. Ensure the surface remains wet for at least 1 minute at room temperature. Then wipe the surface dry.

Once opened, the unused portion of the product may be stored tightly in the tub container for up to 60 days.

***This product is intended for use against HIV-1 in settings where the virus would be expected to be encountered, such as settings where contamination by blood or body fluid is likely.**

Personal Protection: Personnel that clean items soiled with blood or body fluids must wear appropriate personal protective equipment (i.e., barrier protection), such as disposable gloves, gowns, masks or eye coverings

Cleaning: Target surfaces must be cleaned prior to disinfection, because the presence of heavy soil on a surface may reduce the intended disinfectant efficacy of the product.

Infectious Materials Disposal: Any materials used in the cleaning process that may contain blood or body fluids are to be disposed of immediately in accordance with local regulations for infectious materials disposal.

Refer to **Directions for Use**.

Precautions:

May cause eye irritation. Avoid contact with eyes and skin. Use in a well ventilated area. Avoid excessive inhalation of vapours. Keep out of reach of children.

First Aid:

If swallowed, give a glass of water. If contact with eyes or skin occurs, flush thoroughly with water. For excessive inhalation, remove person to fresh air. Get medical attention if ingested, if contact with eyes occurs or if an irritation persists.

Storage:

Flammable. Keep away from heat, sparks and open flames. Store in a well ventilated environment at room temperature.

Disposal:

Rinse the empty refill bag. Do not reuse the empty refill bag. Dispose of the bag in accordance with municipal and provincial requirements. For information on spills and for clean up of spills or the disposal of unused, unwanted product, consult the SDS or contact maxill or the provincial regulatory agency.

maxill inc.

St. Thomas, ON, Canada N5R 6C8

www.maxill.com 1-800-268-8633



< 6070561163RU >

Made in Canada with domestic and imported parts

REV.2016/05