maxill

ECONOMICAL **33x** CONCENTRATE

ULTRASONIC CLEANING SOLUTION SOLUTION DE NETTOYAGE À ULTRASONS

- Economical Concentrate
- Corrosion Protection
- Fresh Lemon Scent
- Concentré économique
- Protection contre la corrosion
- Parfum frais de citron

#60037

Fresh
Lemon Scent

1 L (33.8 FL OZ) 1.06 QT

Ultrasonic Cleaning Solution (UCS³³) is an excellent cleaning solution that will thoroughly lift away buffing compound, blood, serum, pumice and other debris from burrs, scalpels, instruments, diamonds, glass slabs, orthodontic appliances, etc. UCS³³ acts quickly, safely and effectively and is gentle on metals that are being cleaned or on metallic parts of your ultrasonic unit.

Directions: Observe good industrial hygiene practices. Wash hands after handling. Store away from incompatible materials. Use the working solution directly in the ultrasonic unit. Dilute 1-part UCS³³ with 32 parts water, near room temperature. E.g. Mix 58 mL / 2 fl. oz. of UCS³³ with water making up a working solution to 1.9 L / 2 qt. Do not add any other chemicals. Subject the instruments to ultrasonic cleaning for 3 to 5 minutes. Rinse instruments well with water and air dry. For best results, change your ultrasonic bath daily or earlier if solution becomes visibly dirty.

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
maxill inc.
www.maxill.com 1-800-268-8633

Manufacturer:
maxill inc.
St. Thomas, ON
N5R 6C8 Canada

Distributed in USA by:
maxill inc.
Cortland, OH
44410 USA

REF 6070560037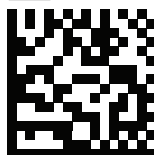

+D89260705600370L

REV.2026/04