#61162

DIN 02342316

**TOWELETTES**Virucidal[†] Bactericidal**LINGETTES**Virucide[†] Bactéricide**ACTIVE INGREDIENTS / INGRÉDIENTS ACTIFS:**

n-Alkyl (50% C14, 40% C12, 10% C16)

dimethyl benzyl ammonium chloride0.3% w/w

**160 WIPES/
LINGETTES****15 cm x 18 cm / 6" x 7"****KEEP OUT OF REACH OF CHILDREN
READ THE LABEL BEFORE USING****GARDER HORS DE LA PORTÉE DES ENFANTS
LIRE L'ÉTIQUETTE AVANT L'UTILISATION****DO NOT FLUSH****WARNING****PRECAUTIONARY STATEMENTS:**

Hazards to humans and domestic animals:
Causes mild skin irritation. Causes eye irritation. Wash hands thoroughly after handling. Do not touch eyes.

FIRST AID:

If skin irritation occurs: Get medical help.
IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical help.

Take the container label or product name and Drug Identification Number (DIN) with you when seeking medical attention.

STORAGE: Keep container tightly closed.

DISPOSAL: This container may be refilled by the consumer with Vinox Towlettes Refill Pouch (REF 6070561162R). Refer to the instructions on the label of the refill pouch. To dispose, recycle this container or discard container in trash. Do not reuse towelette. Dispose of towelette in trash after use. Do not flush in the toilet. For information on disposal of unused, unwanted product, contact the manufacturer or the appropriate municipal/provincial/territorial agency or product stewardship organization.

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See the SDS / Voir la SDS.

Made in Canada with domestic and imported ingredients

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REF 6070561162

+D89260705611620L

Vinoxx towellettes provide low level hard surface disinfection and are designed for use in dental and medical clinics, hospitals, nurseries, nursing homes, institutional kitchens and schools where disinfection and prevention of cross-contamination is of prime importance.

Vinoxx is safe for use on non-porous environmental hard and washable surfaces, such as counters, trays, tables, walls, etc.

Vinoxx is safe for use on non-porous, washable, non-critical medical devices such as bedpans, dental chairs, surgical light handles, etc. Vinoxx will not stain or leave grit or soap scum.

Non-abrasive Vinoxx is formulated for use in daily maintenance programs with a balance of detergents, disinfectants, and malodour ingredients that deliver effective cleaning, disinfecting and malodour control.

If you are uncertain as to the material composition of your surface, confirm compatibility with the manufacturer before proceeding. For plastic and painted surfaces, spot test on an inconspicuous area before use.

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***ORGANISMS CONTROLLED / CONTACT TIMES:** When used as directed on pre-cleaned hard, non-porous non-food contact surfaces, this product is a sanitizer and disinfectant, effective against the following microorganisms with contact time:

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DIRECTIONS FOR USE

This product is not to be used as a terminal sterilant/ high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization/high-level disinfection.

TO START THE FEED THROUGH THE LID:

UnscREW the lid from the canister and remove the protective seal. Pull up a corner of the first towellette from the center of the roll and push 2 - 3 cm through the hole in the lid. DO NOT PUSH FINGER THROUGH THE HOLE! Replace the lid tightly on the canister. Lift the flip-top and pull out the towellettes one at a time. Ensure lid remains closed between uses to prevent moisture loss. Refer to the section **DISPOSAL** for information on refilling this canister.

TO SANITIZE HARD, NONPOROUS, NONFOOD CONTACT SURFACES**

Pre-clean surfaces using a towellette prior to sanitization. Discard. Using Vinoxx Towellettes, thoroughly wet the surface and allow to remain wet for 30 seconds. Then wipe the surface dry.

TO DISINFECT HARD, NONPOROUS SURFACES**

Pre-clean heavily soiled surfaces using a towellette. Discard. Using Vinoxx Towellettes, thoroughly wet the surface and allow to remain wet for the appropriate time indicated for the purpose intended. Use additional towellettes if need to ensure the correct wet contact time. Then wipe the surface dry. Do not contaminate food during use and storage.

**A potable water rinse is recommended after sanitizing or disinfecting if:

- any surfaces may come into direct contact with children at the mouthing stage, or
- any surfaces may come into contact with food or drinking water.

*This product effectively kills Human Immunodeficiency virus 1 (HIV-1) in settings where the virus would be expected to be encountered such as settings where contamination by blood or body fluid is likely.

Special Decontamination Procedures:

Personal protection: Personnel that clean items soiled with blood or body fluids must wear appropriate personal protective equipment (i.e. barrier protection), such as disposable gloves, gowns, masks or eye coverings;

Cleaning: Target surfaces must be cleaned prior to disinfection, as the presence of heavy soil on a surface may reduce the intended disinfectant efficacy of the product; and

Infectious materials disposal: Any materials used in the cleaning process that may contain blood or body fluids are to be disposed of immediately in accordance with local regulations for infectious materials disposal.

"Cleaning Care for Dental Chairs
Call 1-800-268-8633 for more information
www.maxill.com/ca/chaircare

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