

[®]**maxill**

1-800-268-8633
www.maxill.com

Instrument Name:



CAUTION

A new non-sterile instrument is enclosed. Please handle with care. Clean and sterilize prior to use.

1 INSTRUMENT

Enclosed in this package is a high-quality, hand finished dental instrument from maxill. Use only for the designed purpose of the instrument to ensure longevity and edge retention. Take care to ensure no damage comes to the instrument during cleaning, sharpening & sterilization procedures. Do not sterilize in heat over 350°F/177°C or to higher than recommended concentrations of chemicals. To do so will void the instrument warranty. Inspect all instruments prior to and after each use, looking for damage to the tips or the handle. If there is any visible damage to the instrument, remove it from service and call maxill for directions on either having the instrument serviced or replaced.

Thank you for using maxill instruments!

maxill inc.

www.maxill.com 1-800-268-8633

Manufactured for:
USA: maxill inc.
Cortland, OH
44410 USA

Manufacturer:
Canada: maxill inc.
St. Thomas, ON
N5R 6C8 Canada



Made in USA
REV.2017/03