

# Cleaning Instructions for **aquist mini-shields**

(Item 53280)



## **Cleaning**

The aquist Mini Shield reusable frames can be washed with lukewarm water and a non-abrasive, mild soap. Ensure to rinse thoroughly with water after washing to remove any soap residue. Dry completely before disinfection.

## **Disinfection**

The Aquist Mini Shield has reusable frames that can be disinfected with a broad spectrum disinfectant observing the kill times of the disinfectant. Any disinfection process must be preceded by proper cleaning.

**NOTE:** Only the frame can be reused. The plastic shields are disposable. Applying Spaulding's Classification the reusable frames are a non-critical item (do not come into contact with oral mucosa and are not used subgingival) and can be reprocessed via disinfection.