Type I: Fast setting compliant with ISO 21563 specification.

(*) Measurements taken at 23°C with deionised water.

maxill maxsét does not require special treatment before use. Avoid breathing any dusts. Vigorously mix the powder with water.

Variations in temperature or water hardness cause small changes in the working time and initial setting time. Wash, rinse, shake the impression and pour the model. The model is dimensionally stable for up to 7 days. The impression must be stored in a sealed plastic bag without adding anything else. Do not leave the impression in open air or immersed in water. After use keep the bag tightly closed, in a cool, dry place. For best results, store bag in maxill maxset canister (sold separately.)

Excellent results can be obtained with maxill's Type 3 and Type 4 plasters.

Caution:


Federal US law restricts this device to sale by or on the order of a dental professional.



maxill inc.

1-800-268-8633 www.maxill.com Manufactured for: USA: maxill inc. Cortland, OH, USA 44410 Manufacturer: Canada: maxill inc. St. Thomas, ON, Canada N5R 6C8

Made in Italy RFV 2018/09

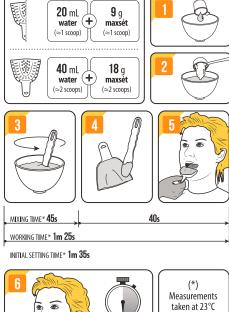



with deionised water.

Item 53776

maxill maxset

High Precision Alginate Storage Canister

> For maximum shelf life, store maxset tightly sealed in its original packaging inside this storage canister.



Net Wt 15.9 oz 450 q