

STARTING WITH A NEW RIBBON HOLDER



Insert the **Ribbon Holder** onto cartridge and turn. Pull the **Ribbon Holder** off the cartridge. The **Transfer Connector** will now be detached from the **Ribbon Holder**. Proceed with steps below.

STARTING WITH A USED RIBBON HOLDER



For best results - Remove air bubbles from each cartridge before transferring materials.

1



Place donor cartridge in the gun and insert a **Transfer Connector** into the tip of the cartridge.

2



Insert the **Ribbon Holder** onto the cartridge, using the arrow indicators to properly align as shown.

3



Connect the recipient cartridge, ensuring that the catalyst and base positions match up.

4



Rotate the **Ribbon Holder** to lock the two cartridges together. Transfer the remaining material to the recipient cartridge.

5

Reverse the rotation of the **Ribbon Holder** to unlock the cartridges. Remove the empty cartridge and dispose of it with the **Transfer Connector**. The **Ribbon Holder** can be reused.



Transfer Connector



Ribbon Holder