

maxill Mixing Bowls

#53771 - Medium (275mL)



#53772- Large (660mL)



SPAULDING CLASSIFICATION: Non-Critical Item; Equipment/device that touches only intact skin and not mucous membranes, or does not directly touch the client/patient/resident

LEVEL OF PROCESSING: Cleaning followed by Low- Level Disinfection

INTENDED USE: For use by dental professionals in the recommended indications. Suitable for mixing alginates, stones, and plasters.

- **NOT** suitable for mixing acrylics or resins.
- **DO NOT** use as transportation system.
- **DO NOT** contaminate interior or exterior of bowls with highly contaminated items such as alginate impressions, or saliva contaminated impression trays.
- **DO NOT** place bowls inside contaminated reprocessing or lab sink

MATERIALS: High quality flexible rubber.

CLEANING (AT POINT OF USE): Wipe the mixing bowl thoroughly with a paper towel to remove debris (hardened stone or alginate). Clean the bowl again with soap and water. If debris is still present, remove debris and repeat cleaning process. Rinse and dry fully before storing.

DISINFECTION: If the mixing bowl has come into contact with saliva or any bio contaminant the use of a broad spectrum disinfectant can be utilized after cleaning process to disinfect mixing bowl.



NOT HEAT TOLERANT- DO NOT STERILIZE



STORAGE: Store in an enclosed area. Do not allow mixing bowl(s) to remain on counter tops in an operatory during procedures when not being used.

***Acceptance for North American Healthcare Facilities**



Spaulding's Classifications

maxill inc.

Canadian Head Office
80 Elm Street
St. Thomas, ON N5R 6C8
Toll Free: 1-800-268-8633
Phone: 1-519-631-7370
Fax: 519-631-3388

USA Head Office
500 West Main Street
Cortland, OH 44410
Toll Free: 1-855-462-9455
Phone: 1-330-637-1176
Fax: 1-330-637-1154