



EARLOOP STYLE PROCEDURAL MASKS

maxill Silken Earloop Procedural Masks are not made from natural rubber latex and offer Level 3 protection*. The anti-fog foam strip provides added comfort and helps keep your glasses fog free.

*ASTM LEVEL 3

Complies with ASTM F2100-19 Requirements:

Fluid Resistance: 160 mm Hg

Bacterial Filtration Efficiency: BFE $\ge 98\%$ **Particle Filtration Efficiency:** PFE $\ge 98\%$ **Breathability - Delta P:** $< 5.0 \text{ mm H}_2\text{O/cm}^2$

Flame Spread: Class 1

Availability

Blue	53061
Pink	53062
Green	53063
White	53064

(50 masks per box)



How to Wear an Earloop Mask



Use your fingers to open the ear-loops. Position the nose clip at the top. Place the mask over your nose and mouth.



Position the ear-loops around your ears. Pull the mask from the top and bottom to unfold the pleats of the mask.



Form the nose clip over the bridge of your nose to make it fit snugly.

Made in China