

SCHIFF'S CHAIR CLEANER & POLISH may be used on non-porous environmental surfaces and smooth upholstery for a streak free shine. Its formulation is gentle and safe for use on leather & vinyl upholsteries, stainless steel appliances and fixtures, Formica or stone-type solid counters and tables, electronics, plastics and more.

SCHIFF'S CHAIR CLEANER & POLISH does not contain mineral oil, is low in VOCs and its non-flammable formula helps to prevent static build-up.

Directions:

Remove any buildup of dirt and grime. Spray **SCHIFF'S CHAIR CLEANER & POLISH** across the surface. With a soft cloth or paper towel, wipe the surface until dry. Repeat cleaning as needed to maintain a clean, streak-free, polished surface.

Manufactured for:

maxill inc.

Cortland, OH, USA 44410
1-800-268-8633 www.maxill.com

Made in Canada

maxill

ITEM 60021

SCHIFF'S

CHAIR CLEANER & POLISH

SPECIALLY FORMULATED FOR USE TOGETHER WITH mycolio™

DENTAL CHAIRS • LAB EQUIPMENT • WORK SURFACES

**BUFFS TO A
STREAK-FREE
SHINE**



Warning
Causes eye irritation

Prevention:

Wash from skin thoroughly after handling.

Response:

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do so. Continue rinsing.
If eye irritation persists: Get medical advice/attention.

Storage:

Protect from freezing or cold.
Store above 39°F at room temperature.

Cleaning Care for Dental Chairs

In addition to disinfecting, a cleaning protocol is required by the chair manufacturer. To learn more about how **SCHIFF'S CHAIR CLEANER & POLISH** can be used to help prolong the life of your dental chair upholstery, call 1-855-462-9455 or visit www.maxill.com/us/chaircare

REV.2019/01



32 FL OZ